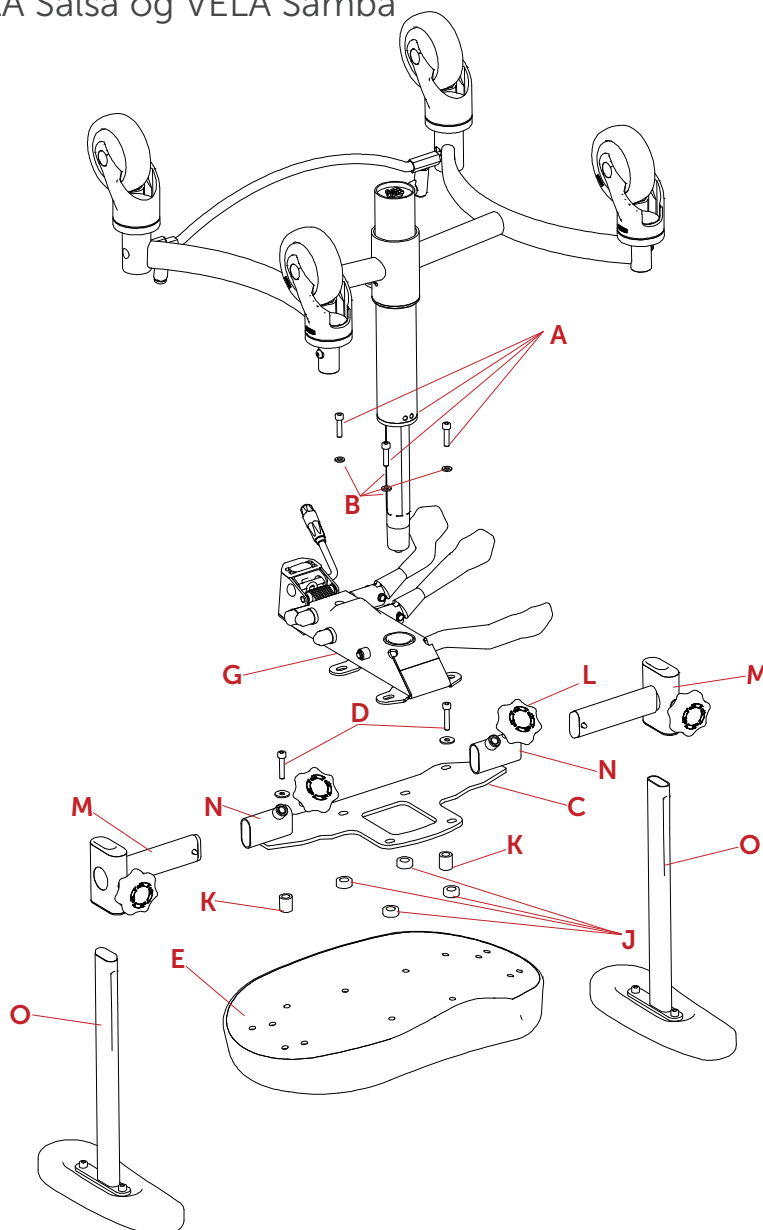
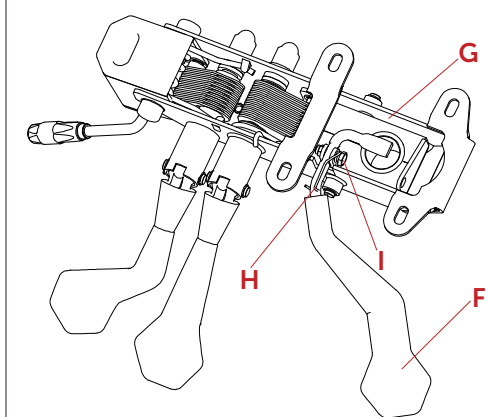
# Sæde & armlæn / Sete & armlene / Sits & Armstöd / Seat & armrest - VELA Salsa/Samba

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### ENGLISH

1. Remove the seat plate by unscrewing the 4 screws (A) and removing the washers (B).
2. If present, the armrest base (C) is removed by unscrewing the 2 screws with washers (D). The seat (E) can now be removed.
3. Only when switching to **a saddle seat**: Replace the release handle (F) for height adjustment on the seat plate (G) by prying off the spring (H) with a small screwdriver and removing the bolt (I). Mount the new handle, install and tighten the bolt (I), and push the spring (H) back into place using a small screwdriver. The four thick washers (J) should be placed between the armrest base (C) and the saddle seat. Screws (D) and spacers (K) are not to be used. **Proceed to step 5.**
4. Screw the armrest base (C) to the seat (E) using the 2 screws (D) and spacers (K). The hand screw (L) must face forward toward the front of the seat. The four thick washers (J) are not to be used.
5. Place the seat plate (G) onto the armrest base (C) and seat (E), and fasten it with the 4 screws (A) using washers (B) underneath. Tighten firmly.
6. Mount the side parts (M) into the armrest bracket (N), and the armrest bars (O) into the side parts (M).

Test the release handle (F). If it interferes with the hand screw (L), gently pull the handle slightly forward.