# Udskiftning af gaspatron/ Skifte gasspatron/ Byte av gaspatron/ Replacing the gas cartridge - VELA Tango 500/510


VELA Tango 500/510

Varenummer/ Artikelnummer/ Product number:

(A)928306 Gas 350N 150mm (varenr u/alu hus 093035151)

(A)928307 Gas 350N 200mm (varenr u/alu hus 093035201)

(A)928308 Gas 500N 150mm (varenr u/alu hus 093050151)

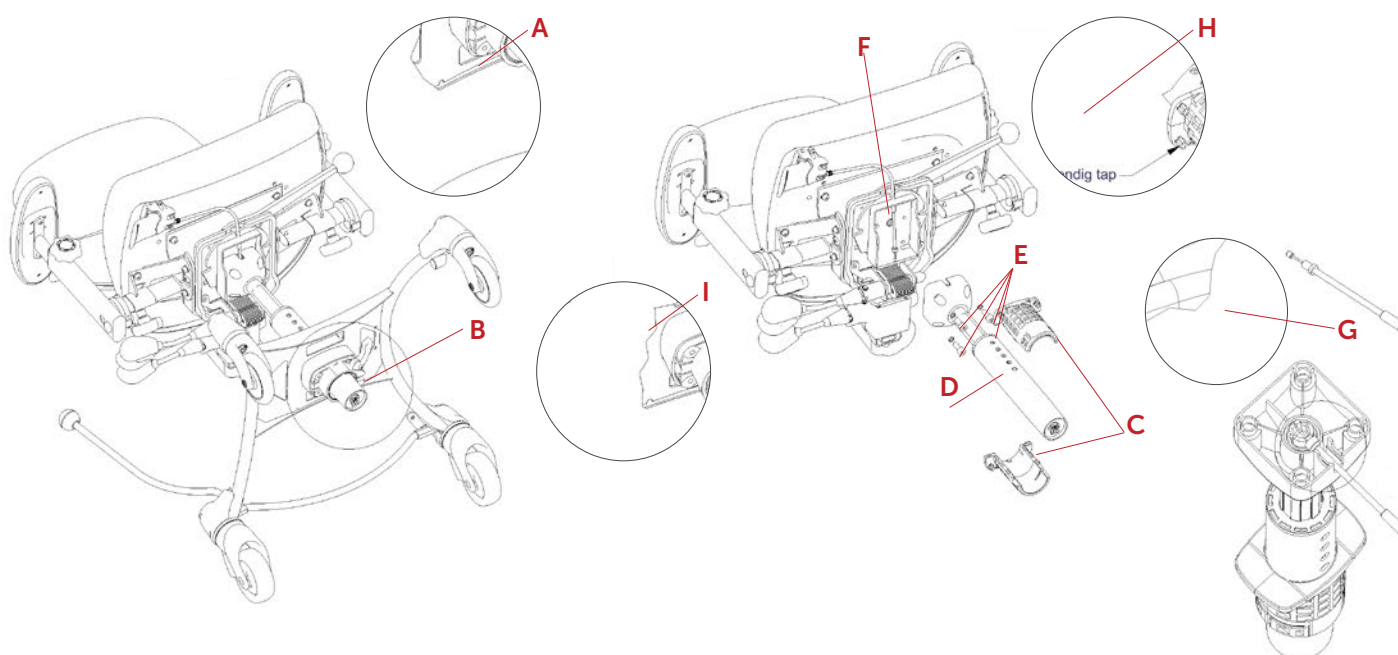## DANSK

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#### ENGLISH

1. Pull the handle at the bottom of the frame (A).
2. Remove the underframe by carefully hitting the bottom of the external tube with a rubber mallet (B).
3. Remove the 2 plastic pieces from the gas cartridge (C). Mark the holes (D) where the 2 plastic pieces were located in order to position the new gas cartridge at the same height.
4. Remove the gas cartridge with fixture by unscrewing the 4 screws beneath the seat (E) and removing the release wire (F). If necessary, use a pair of needle nose pliers for this step.
5. Place the release wire in the fixture on the gas cartridge (G), and turn the fixture so that the small groove is facing forward. Use the 4 screws (E) to screw the fixture in place beneath the seat.
6. Set the 2 plastic pieces (C) on the new gas cartridge with the fixture. Make sure that the internal pin on both plastic pieces is fully inside the hole on the gas cartridge (H).
7. Set the entire top piece down in place in the underframe. Check the external pins on the plastic pieces to ensure they fit in the holes in the underframe.
8. Place a knee on the seat and use your body weight to press the top piece all the way into place in the underframe, and then lock the handle at the bottom of the frame (I).