Gasfjeder / Gas spring / Gassfjær - VELA Tango 700

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ENGLISH

Interval adjustment is done by following point 1-3 and 7-8.

1. Remove the covers (A) by removing o-rings (B) and pull them out to the side. If necessary use a small screwdriver.
2. Slide the lower cover upwards (C), until the two M8 setscrews (D) appear behind the column. Use a 4mm hex key to loosen them.
3. Lift the upper part of the chair (E) out of the chassis (F) and remove the two plastic bushings (G).

Note which hole (H) in the gas cartridge they are in, to mount the new gas spring at the same height.

4. Remove the gas spring (I), by loosening the 4 M8 screws (J) with a 6 mm hex key.

5. Remove wire (K). If necessary use nose pliers for this. Remove the bottom cover (L). Remove the holder (M) from the gas spring (I) with rubber hammer.
6. Fit the new gas spring into the holder. This is a tapered assembly. Pay attention to the orientation of the gas spring in relation to the holder as incorrect orientation can skew the chair.
7. Put the two plastic bushings (G) on again. Remember to orientate them correctly depending on the wheel type on the chair.
8. Assemble the rest by doing the above in reverse order. The two M8 setscrews (D) at the back of the column must be tightened with 7 Nm.

Note: (N) Facing down on single wheels, (N) facing up on twin wheel.