



Assembly manual for VELA Meywalk 2000 - model Medium and Large






Assembly manual for VELA Meywalk 2000 - model Medium and Large




Mount rear wheels: Put bolt through wheel, slide bolt through under frame, screw nut on, put center cap on.

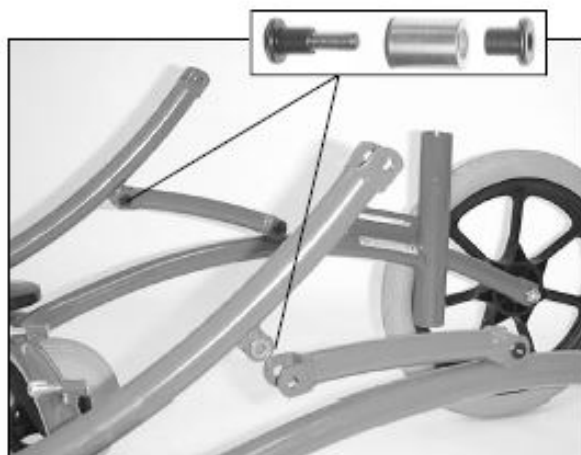

Mount tilt bar on under frame: Place bushes in hinge eyes on tilt bar, screw together with Allen screw heads and threaded rods.




Slide down tubes into under frame.

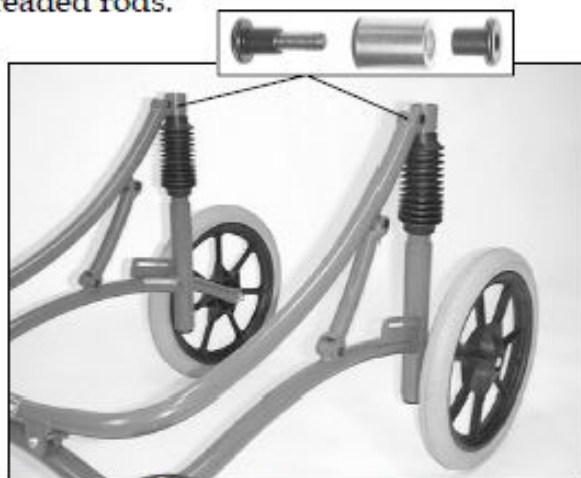

Mount tilt bar on down tubes: Place bushes in hinge eyes on down tubes, screw together with Allen screw heads and threaded rods.



Assembly manual for VELA Meywalk 2000 - model Medium and Large

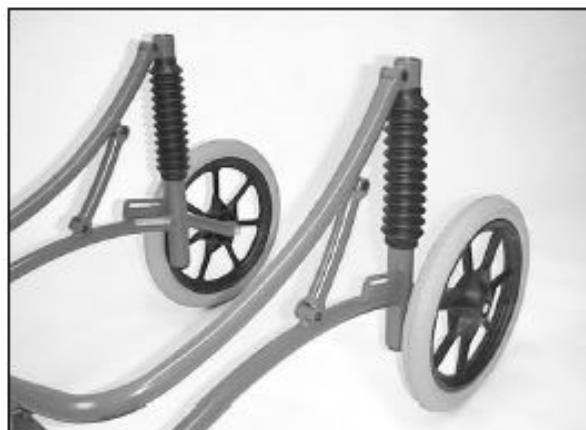

Raise tilt bar. This will push the rubber covers down to their correct position.
Lower tilt bar again.



Mount brakes: Put bolts through slit on each side of under frame, put brake mechanism onto bolts, put screw nuts on.



Adjust brakes so there is a gap of 3-5 mm between tyre and brake fitting.



Mount trunk support: Trunk support and plastic washers are placed on top of top part, Allen screws with washers are screwed in from underside.



Assembly manual for VELA Meywalk 2000 - model Medium and Large




Mount spanner grips: Place first U-shaped washer and then plastic washer on spanner grip, screw spanner grip on.



Mount seat: Slide seat fitting onto seat tube, tighten Allen screw.



Mount rear stop: Slide rear stop tube into seat tube, tighten Allen screw.

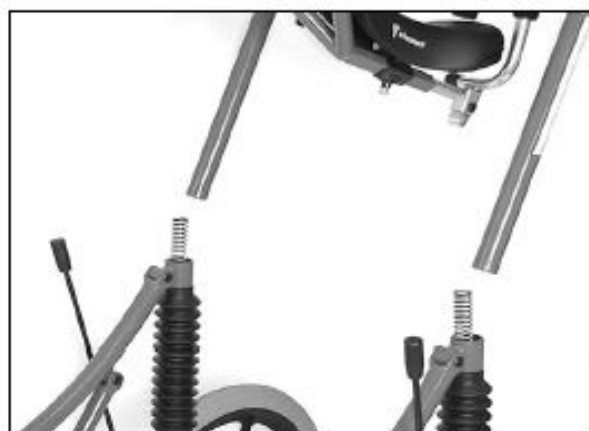

Mount top part on under frame: Place springs in top of down tubes. Slide top part into down tubes.