





# E-fodstøtter/ E-fotstøtter/ E-benstöd/ E-footrests - VELA Tango 700

## VELA Tango 700

### NORSK

1. Hvis stolen har seterotasjon, skal denne fjernes ved å demontere håndtaket (A). Rotasjonen låses (se anvisning 106247). Det kan være nødvendig å ta av deksler for å komme til (se anvisning 106231).
2. Høydejusteringshåndtaket (B) demonteres ved å fjerne de to skruene (C) (Gjelder ikke elektrisk justerte stoler).
3. De to fremre vingskruene (D) under setet demonteres.
4. De to beslagene (929342+929343) (E) monteres med tilhørende skruer og skiver (F) (2 x 3 stk.).
5. Høydejusteringshåndtaket (B) monteres på setet med de lengre skruene (C) gjennom plastdelen (G) (Gjelder ikke elektrisk justerte stoler).
6. Skyvestangen (928245) (H) monteres i C-profilen på brakett 929342/929343 (E). Stram skyvestengene (H) godt fast i ønsket posisjon.
7. E-fotplater (I) senkes ned i skyvestangene (H).

### SVENSKA

1. Om stolens säte har rotationsfunktion kan du ta bort det genom att avmontera handtaget (A). Rotationsfunktionen låsas (se anvisning 106247). Du kan behöva montera av skyddet för att komma åt (se anvisning 106231).
2. Höjdinställningshandtaget (B) demonteras genom att ta bort de två skruvarna (C) (Gäller ej stolar med el).
3. De två främre vingmuttrarna (D) under sätet demonteras.
4. Montera fast de två fästena (929342 + 929343) (E) med tillhörande skruvar och brickor (F) (2 x 3 stycken).
5. Höjdinställningshandtaget (B) monteras på sitsen med de längre skruvarna (C) genom plastdelen (G) (Gäller ej stolar med el).
6. Skjutstången (928245) (H) monteras i C-profilen på beslag 929342/929343 (E). Dra åt skjutstångerna (H) ordentligt i önskad position.
7. E-fotstöden (I) sänks ned i skjutstångerna (H).

### ENGLISH

1. If the chair has a seat rotation, remove it by removing the handle (A). Fixate the rotation (inst. 106247). You may need to remove covers (inst. 106231).
2. The height adjustment handle (B) is removed by taking out the two screws (C) (Not applicable for chairs with electric adjustment).
3. The two front wing screws (D) under the seat are removed.
4. Mount the two brackets (929342 + 929343) (E) with the corresponding screws and washers (F) (2 x 3 pieces).
5. The height adjustment lever (B) is mounted to the seat with the longer screws (C) through the plastic part (G) (Not applicable for chairs with electric adjustment).
6. The slide rod (928245) (H) is mounted in the C-profile of bracket 929342/929343 (E). Tighten the slide rods (H) securely in the desired position.
7. E-footrest (I) are mounted in the slide rod (H) as shown.